



The brace helps dogs suffering from hip dysplasia, reducing pain by exerting bilateral compression on the hips. It is effective for all degrees of dysplasia including degree E (the most severe). It is also used for other pathologies (hip arthrosis for example). It is an excellent alternative to surgery in cases where the dog cannot be operated. It is made of breathable and washable material that is both retentive and compressive.

BT LIFE

comes with 2 handles adjustable in length that can be hooked to the brace in case of need and that the owner can use to help the dog get into or out of a car or on stairs. The handles can be removed if they are not necessary.

ADVANTAGES

BT Life consists of 2 rigid disks lined with a double layer of soft high density material. These disks act to compress both hips, increasing body temperature in the zone and thus reducing the pain caused to the dog by the pathology. Constant use of the brace leads to concrete well-being in the life of the dog, reducing part of the effects of dysplasia.

FEATURES


- ▶ Reduced pain thanks to bilateral compression of the hips.
- ▶ Breathable fabric.
- ▶ Easy fit.
- ▶ Hand washable.
- ▶ Adjustable straps and handles.

MEASURES

Size	Groin Circumference	Breeds and weights*
XS	15-20 cm	Jack Russel , Shih Tzu (5-10 kg)
S	25-30 cm	Cocker Spaniel (10 -15 kg)
M	30-35 cm	Border Collie, Setter (15 -30 Kg)
L	40-45 cm	Golden Retriever, Rottweiler (28 -50 Kg)
XL	50-55 cm	Saint Bernard, Great Dane, Newfoundland (Over 50 Kg)

***Attention the suggested breeds and weights are purely indicative, to determine the size of the brace take the measurement as specified.**

HOW TO TAKE THE MEASURE

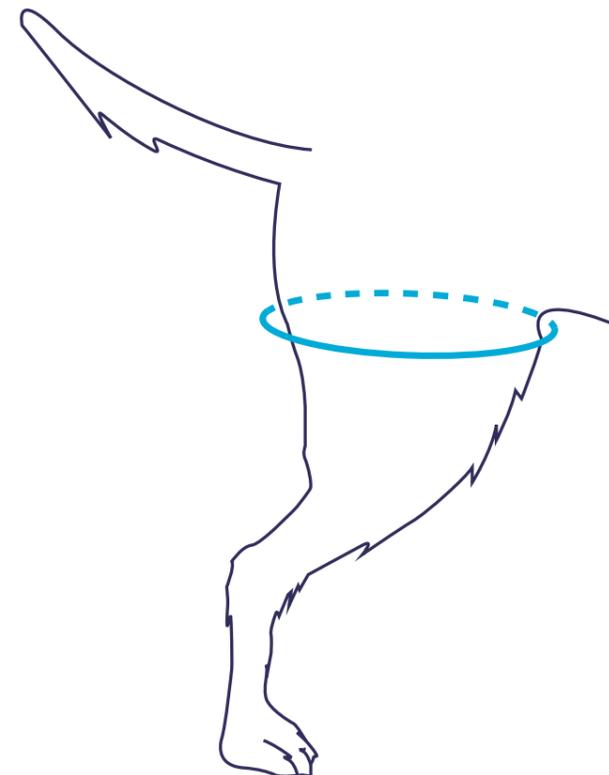

Measure the circumference of the leg at the level of the groin with a tape measure.

